


Technical data:

PARAMETER:	Model AR-101	Model AR-104
Frame (material):	steel	aluminum
Seat width:	50 cm	43 cm
Seat depth:	37 cm	42 cm
Toilet seat width:	35 cm	not applicable
Toilet seat depth:	37 cm	not applicable
Height adjustment:	40-55 cm	42-52 cm
Folded thickness (without container)	15 cm	15 cm
Sanitary container capacity:	7 liters	5 liters
Product weight:	6kg	4kg
Maximum user weight:	136 kg	120 kg

SYMBOLE:

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature
Purchase invoice number	
Model / LOT No	

1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a 12-month warranty for the products covered by this document. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, i.e. scratches, scratches on plastic and rubber elements (i.e. abrasion of rubber caps (8), plastic armrests (3))
6. The warranty does not cover if the chair is used contrary to its intended purpose or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
8. **The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.

The advertised product must be in proper hygienic condition.



**USER MANUAL
WARRANTY CARD**

TOILET CHAIR – FOLDABLE

Model: AR-101

**TOILET AND SHOWER CHAIR
ALUMINUM – FOLDABLE**

Model: AR-104



Model: AR-101



Model: AR-104

ARMEDICAL Sp. z o.o.
 al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
 tel.fax: +48 32 261 71 36, +48 783 001 111
 www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - TOILET (SHOWER) CHAIR – FOLDABLE

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will significantly facilitate your daily toileting activities. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The toilet, toilet and shower chair is intended for disabled people with musculoskeletal disorders, elderly people as well as people who have problems using a standard toilet bowl. As a toilet, it allows you to meet your physiological needs without having to get up from your chair. As a shower chair (applies to AR-104), it is dedicated to people for whom standard use of a bathtub or shower cabin is a problem. It facilitates and helps with everyday toilet activities, e.g. at the sink, in the bathtub or shower. The toilet chair (applies to AR-101) can also be used as handrails to facilitate getting up from a standard toilet bowl (after removing the sanitary container (4) and assembling the toilet seat (5) and the toilet seat cover (2)).



Contraindications:

Contraindications to a sitting position, lack of trunk stabilization. Movement coordination and balance disorders, labyrinth problems, spinal instability. Intellectually limited people should not use the product without the supervision of a guardian.

Usage data:

Maximum patient weight ensuring safe use of the product – [AR-101 - 136 kg] [AR-104 - 120 kg]



Notes and warnings:

To avoid an accident, please:

1. Check whether all tubes are straight and not broken (e.g. during transport)
2. Checking whether all plastic elements have no damage, e.g. cracks
3. Check whether all chair legs are set at the same height
4. Check whether there are rubber pads on all legs and whether they are all installed correctly
5. Do not unfold the chair in places where it may be slippery (spilled liquid, ice)



To avoid injuries:

1. The toilet chair (AR-101) should be fully unfolded (the locking latches in the crossbars must "click" into their appropriate holes). Regarding AR-104 - the safety latches must be completely lowered and pressed onto the locking sleeves. The entire surface of the sleeve must adhere to the recesses of the locking pawls
2. It is necessary to check the technical condition and stability of the product each time before use (checking all fastening elements, product frame tubes, rubber caps and toilet seat/seat).
3. All chair legs must be at the same height
4. Use only on a flat, hard surface that prevents the chair from moving when sitting down
5. Be especially careful on wet or slippery surfaces.
6. Sitting on a chair abruptly (including jumping) is prohibited.
7. It is forbidden to move the chair together with the user
8. It is forbidden to stand with shoes on armrests, toilet seats or any element of the product.
9. Do not overload the product or exceed the maximum load specified by the manufacturer.
10. You should only sit on the chair (AR-101) with the toilet seat lowered and the toilet seat cover open (raised). Sit only when the seat with the hole is lowered - (applies to AR-104)

Maintenance method:

Each time after toileting, the product should be thoroughly washed and rinsed with clean water (AR-104 - do not leave any soap, shampoo, bath gel, etc. on the product) and then dried thoroughly with a delicate cloth. The use of strong detergents, gasoline and solvents is prohibited. Failure to comply with hygiene rules will result in loss of functional and aesthetic parameters and thus loss of warranty. Do not use abrasive products! The leg caps (8) are wearing parts and must be replaced regularly. **Let's remember! The product is a toilet chair, so hygiene rules must be particularly observed.**

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the chair yourself is FORBIDDEN!!! This may result in an accident.

I. The chair is not stable.

- a. Are all chair legs at the same height? If not, set them correctly.
- b. Do all legs have rubber caps (8)? If not, put them on.
- c. Are all rubber caps (8) installed correctly? If not, slide them on securely.
- d. Is the chair fully reclined? If not, attach the transverse beams in such a way that the locking latches "click" into their appropriate holes - applies to AR-101. The chair must be fully unfolded, the safety catches must be fully lowered and pressed onto the locking sleeves - AR-104
- e. Is the toilet seat securely attached (5)? If not, press it carefully so that it is stably attached to the chair frame - regarding AR-101
- f. Are all frame elements stably connected to each other? If not, please contact your point of purchase. DO NOT USE A TOILET CHAIR!!!

Preparation for use

1. Remove the chair from the cardboard or foil packaging.
2. First, check the technical condition of the product in accordance with point **Notes and warnings**.
3. Regarding AR-101 - grab the plastic armrests (3) with both hands and move the armrests outwards until the chair is unfolded. Install the front beam (6) into the leg holes. Check whether the locking latches "click" into their appropriate holes. Install the rear beam (1) supporting the toilet seat cover (2). Check whether the locking latches "click" into their appropriate holes. Insert the sanitary container (4) and then lower the toilet seat (as shown in the drawing above). Regarding AR-104 - grab the front and rear frame legs (2) with both hands, spread them as far as possible, secure with the safety catches (5) (the safety sleeves must be located in the notches of the safety catches). Insert the sanitary container (4) and then lower the seat (6) (as shown in the drawing above).
4. Adjust the height of the chair legs to the user's height by pressing the head of the latch (7) located in the leg, which will release the lock. Set the required height by extending or sliding the tube. The latch head must engage in the hole at the desired height! Repeat the operation for all legs. The latch head on all legs must be at the same height.

UTILITY ELEMENTS



Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Assemble the chair (AR-101) by dismantling the beams (1) and (6) and pressing the locking latches, remove the sanitary container and then move the handrails inside.
2. Fold the chair (AR-104) by unlocking the safety catches (5), and then slide the aluminum frame legs (2) towards each other. The chair can be folded "with" and "without" a sanitary container. To remove the sanitary container, lift the seat towards the backrest.

Cleaning and storage:

1. Place the chair in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.

Incidents:

Every **serious** incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT