










Technical data:

PARAMETER:	Model AR-102	Model AR-103
Seat width:	42 cm	45 cm
Seat depth:	45 cm	42 cm
Width between armrests:	47 cm	45 cm
Depth with seat cover:	45 cm	45 cm
Height to the seat:	43 – 57 cm	42.5 – 57.5 cm
Total external dimensions:	width 50 cm x depth 62 cm x height 85 cm	width 60 cm x depth 54 cm x height max. 96 cm
Width (thickness) after folding:	14 cm	not applicable
Sanitary container capacity:	4 liters	5 liters
Product weight:	7.2kg	7kg
Maximum user weight:	120 kg	120 kg

SYMBOLE:

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD
The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature
Purchase invoice number	
LOT No	

1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a 12-month warranty for the products covered by this document. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only within the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors (e.g. frame upholstery, cuts, tears in upholstered elements and any damage resulting from improper use/cleaning/storage of the toilet chair). The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, i.e. scratches, scratches on plastic and rubber elements (i.e. abrasion of rubber caps, plastic armrests, upholstered elements)
6. The warranty does not cover if the toilet chair is used contrary to its intended purpose or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
8. **The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.
The advertised product must be in proper hygienic condition.



**USER MANUAL
WARRANTY CARD**

UPHOLSTERED TOILET CHAIR
Model: AR-102 FOLDABLE
Model: AR-103



Model: AR-102



Model: AR-103

ARMEDICAL Sp. z o.o.
al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - UPHOLSTERED TOILET CHAIR

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will significantly facilitate your daily toileting activities. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Purpose and description of the product:

The toilet chair is intended for disabled people with musculoskeletal disorders, elderly people and people who have problems using a standard toilet bowl. It facilitates and helps with everyday toileting activities and allows you to meet your physiological needs without having to get up from your chair. The upholstered toilet chair is intended mainly for people who are sensitive to pressure and feel discomfort when using a standard toilet chair, and for people who value comfort and a high level of hygiene. Particularly recommended after hip joint injuries. The soft seat and backrest ensure extraordinary comfort of use.

⚠️ Contraindications:

Contraindications to a sitting position, lack of trunk stabilization. Movement coordination and balance disorders, labyrinth problems, instability spine. Intellectually limited people should not use the product without the supervision of a guardian.

Usage data:

Maximum patient weight ensuring safe use of the toilet chair –120 kg

⚠️ Notes and warnings:

To avoid an accident, please:

1. Check whether all tubes are straight and not broken (e.g. during transport)
2. Check whether all structural elements have no damage, e.g. cracks or breaks
3. Check whether all legs of the toilet chair are set at the same height
4. Check whether there are rubber pads on all legs and whether they are all installed correctly
5. Do not unfold the toilet chair in places where it may be slippery (spilled liquid, ice) - regarding AR-102

⚠️ To avoid injuries:

1. The toilet chair should be fully unfolded (refers to AR-102) (the legs of the chair must be fully open, unfolded to the full stop). The chair must look like in the photo.
2. It is necessary to check the technical condition and stability of the product each time before use (checking all fastening elements, seat, product frame tubes, rubber caps)
3. All legs of the toilet chair must be set at the same height
4. Use only on a flat, hard surface that prevents the chair from moving when sitting down
5. Be especially careful on wet or slippery surfaces.
6. Sitting on a chair abruptly (including jumping) is prohibited.
7. It is forbidden to move the chair together with the user
8. It is prohibited to stand with shoes on the armrests, seat or any element of the product.
9. Do not overload the toilet chair, exceeding the maximum load specified by the manufacturer.
10. The toilet chair should be used only for its intended purpose specified in this manual. Using the toilet chair for any other purpose may result in an accident, injury or other consequences for the user or third parties. The manufacturer is not responsible for using the toilet chair for purposes other than those described in this manual.

Maintenance method:

The upholstered toilet chair should be cleaned regularly using only water and mild detergents (washable material). Then dry thoroughly with a soft cloth. The use of strong detergents, gasoline and solvents is prohibited. Storage in rooms with high humidity, e.g. (basement), combined with lack of thorough product maintenance (hygiene) will result in the loss of the functional and aesthetic parameters of the chair and thus the loss of the warranty. Do not use abrasive products!

Let's remember! The product is a toilet chair, so hygiene rules must be particularly observed.

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing a toilet chair yourself is FORBIDDEN!!! This may result in an accident.

I. The toilet chair is not stable.

- a. Are all chair legs at the same height? If not, set them correctly.
- b. Do all legs have rubber caps (7)? If not, put them on.
- c. Are all rubber caps (7) installed correctly? If not, press/slide them in well.
- d. Is the chair fully unfolded (applies to AR-102)? If not, arrange them correctly. Spread the front and rear legs apart (until they stop).
- e. Does the seat (2) firmly rest on all edges of the chair frame (regarding AR-103)(5)? If not, position it correctly on the frame.
- f. Are all frame elements stably connected to each other? If not, please contact your point of purchase. DO NOT USE A TOILET CHAIR!!!

Incidents:

Everyone serious an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

UTILITY ELEMENTS



1. Upholstered backrest
2. Upholstered seat
3. Upholstered opening cover
4. Armrests
5. Frame
6. Sanitary container
7. Rubber caps
8. Height adjustment latches
9. Height adjustable legs

Preparation for use

1. Remove the toilet chair from the cardboard or foil packaging.
2. First, check the technical condition of the product in accordance with point **Notes and warnings**.
3. Grab the backrest (1) and seat (2) with both hands and unfold the chair until you feel resistance (applies to AR-102) - when unfolded, it must look like in the photo. Insert the sanitary container (6) under the seat (2) - from the front (for AR-102). The AR-103 toilet chair does not require disassembling the frame. Insert the sanitary container into the holders under the seat after removing the seat (2) (applies to AR-103). Place the seat (2) on the frame structure (5) of the toilet chair (applies to AR-103).
4. Adjust the height of the legs (9) of the toilet chair to the user's height by pulling out the height adjustment latch (8) located in the leg, which will release the lock. Set the required height by extending or sliding the tube. The latch pin must engage in the hole at the desired height! Check whether the height adjustment latch (8) "clicks" into the correct hole. Repeat the operation for all legs. ATTENTION! The latch pin on all legs must be at the same height. All four legs (9) of the toilet chair must be at the same height.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Regarding AR-102 - remove (pull out) the sanitary container (6). Fold the toilet chair by grasping the seat (2) and backrest (1) with both hands. Lift the seat towards the backrest until the chair is completely folded. For AR-103, remove the sanitary container (6), place the upholstered cover of the opening (3) on the frame.

Cleaning and storage:

1. Place the toilet chair in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT