


Technical data:

PARAMETER:	Model AR-206	Model AR-207
Material:	polypropylene	aluminum / polypropylene
Seat dimension:	75x35cm	41x22 cm
For the width of the bathtub:	40-62 cm	52-61 cm
Libra:	2.7kg	1.5kg
Maximum user weight	160 kg	110 kg

SYMBOLE:								
								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature
Purchase invoice number	
Batch No	

1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a 12-month warranty for the products covered by this document. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only within the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, i.e. scratches, scratches on polypropylene and plastic elements (i.e. their abrasion during operation).
6. The warranty does not cover if the bathtub bench is used contrary to its intended use or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
8. **The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded. **The advertised product must be in proper hygienic condition.**



**USER MANUAL
WARRANTY CARD**

BATH BENCHES

Model: AR-207 recessed
Model: AR-206 with handle



Model: AR-206



ARMEDICAL Sp. z o.o.
al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - BATH BENCHES

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will significantly facilitate your daily toileting activities. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The bathtub bench with a handle/recessed as an auxiliary device is intended for disabled people with disorders of the musculoskeletal system, the elderly and people for whom standard use of the bathtub is a problem for health reasons. It facilitates and helps with everyday toileting activities and at the same time increases the safety of disabled people who want to do toileting in the bathtub.



Contraindications:

Contraindications to a sitting position, lack of stabilization of the trunk and head. Movement coordination and balance disorders, labyrinth problems, spinal instability. Intellectually limited people should not use the product without the supervision of a guardian.

Usage data:

Maximum patient weight ensuring safe use of the bench – AR-206 –160 kg, AR-207 –110kg



Notes and warnings:

To avoid an accident, please:

1. Check whether the bench and handle(s) do not have any obvious mechanical damage (e.g. during transport)
2. Check that all mounting screws/latches of the handle(s) are securely tightened
3. AR-206 checking whether the bench has all 4 expansion legs (2), AR-207 all 4 handles (1)
4. AR-206 checking whether the plastic expansion legs (2) wedge against the sides of the bathtub, preventing it from moving. AR-207 checking whether all 4 handles rest on the edges of the bathtub with their entire surface.
5. Check whether the spacing adjustment latches (2) (pins) are firmly attached.



To avoid injuries:

1. It is necessary to check the technical condition and stability of the product each time before use (all screwed elements, i.e. handle(s) and expansion legs)
2. AR-206 The handle of the bathtub bench (1) must be solidly and permanently attached (using the attached screws) to the seat (3) AR-207 Adjustable handles (1) spacing must be firmly locked with latches (2) (pins) in the tubes located under the seat
3. AR-206 The expansion legs (2) must be solidly and permanently attached (using the included screws) to the seat
4. Use on the bathtub only in a horizontal position, after previously "wedging" the bench into the bathtub using expansion legs (2). Once installed, the bench cannot move freely around the bathtub. AR-207 after adjusting the width of the handles so that their entire surface rests on the edges of the bathtub.
5. Be especially careful when using it, do not tilt backwards suddenly
6. It is forbidden to sit down abruptly on the bench (including jumping)
7. It is forbidden to move the bench together with the user
8. It is forbidden to stand with shoes on the bench or any element of the product.
9. Do not overload the product or exceed the maximum load specified by the Manufacturer.
10. It is recommended that the bottom of the bathtub be lined with anti-slip mats

Maintenance method:

Each time after toileting, the product should be rinsed thoroughly with clean water (do not leave any soap, shampoo, bath gel, etc. on the product, and then dried thoroughly with a soft cloth. It is prohibited to use strong detergents, gasoline or solvents for cleaning. Storage in rooms with high humidity (e.g. basement) combined with lack of thorough maintenance of the product (hygiene, drying) will result in loss of functional and aesthetic parameters of the bathtub bench, and thus loss of warranty. Do not use abrasive agents!

Troubleshooting:


If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the product yourself is FORBIDDEN!!! This may result in an accident.

I. The bathtub bench is not stable.


- a. Are the bench handle(s) (1) securely screwed to the seat (3)? If not, attach them.
- b. AR-206 Are all expansion legs (2) securely screwed to the seat (3)? If not, tighten them.
- c. AR-206 Do all expansion legs (2) adhere to the side edges of the bathtub, preventing it from moving? If not, they should be moved apart so that they prevent the bench from moving freely on the bathtub; AR-207 – do all adjustable handles lie with their entire surface on the side edges of the bathtub? If not, move them apart.
- d. All product elements should be assembled/assembled in such a way that, when completed, they constitute an exact reproduction of the product photo below
- e. Despite firmly tightening all elements with mounting screws, the product is still not stable? If not, please contact your point of purchase. DO NOT USE A BATH BENCH!!!

COMPONENTS/UTILITY ELEMENTS

Model: AR-207

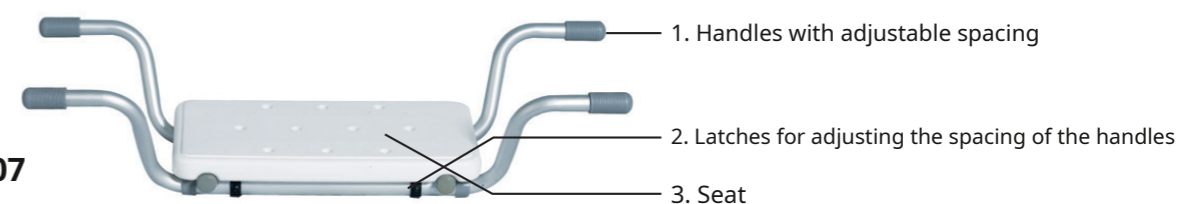

Model: AR-206

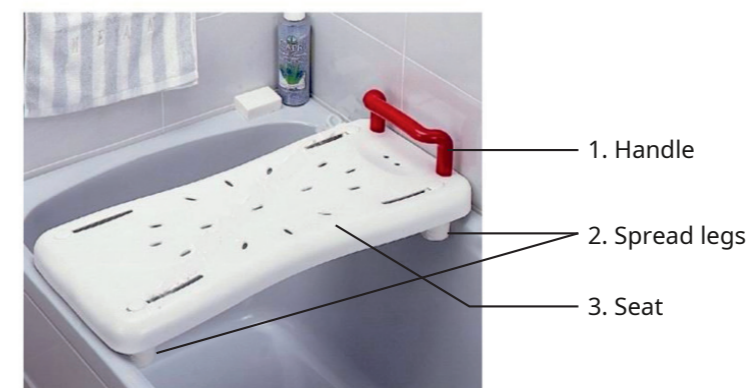

Preparation for use:

1. Remove the bathtub bench from the cardboard or foil packaging. The product removed from the packaging requires assembly before first use.
2. First, check the technical condition of the product in accordance with point **Notes and warnings**.
3. Place the AR-206 seat (3) on the floor or a hard surface so that the upper part is visible. Grab the handle (1) and place it on the seat (3) as shown in the drawing above, so that the holes in the handle and seat line up with each other. Using a screwdriver (screwdriver), insert the included screws into the existing holes. After assembling the seat (3) and the handle (1), screw on the expansion legs (2). Lift the seat slightly and insert all 4 expansion legs (in places as shown in the drawing), then screw in the included screws from above. After placing the bench on the bathtub, move the expansion legs outwards so that they "wedge" the bench in the bathtub, preventing it from moving. After properly adjusting the spacing of the expansion legs, firmly tighten the screws securing the legs. AR-207, remove the spacing-adjustable handles (1) and insert them into the individual tubes located under the seat (3). Each handle (1) must be locked with a latch for adjusting the spacing (2) (pin inserted into the through hole in the tube). Adjust the spacing of the handles (1) to the width of the bathtub.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Place the bathtub bench in a place where it will not be exposed to mechanical damage; the product can also be left on the bathtub.

Cleaning and storage:

1. In case of dirt, follow the "Maintenance method" section of this manual.

Incidents:

Everyone **serious** an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT