









**Technical data:**

PARAMETER:	Model AR-001	Model AR-002	Model AR-003	Model AR-008	Model AR-009	Model AR-023
Height adjustment:	79-96 cm	79-96 cm	70-83 cm	70-83 cm	78-85 cm	79-89 cm
Frame width walker after unfolding:	50 cm	50 cm	60 cm	50 cm	50 cm	57 cm
Width (thickness) walker after folding:	10 cm	10 cm	10 cm	10 cm	20 cm	10 cm
Libra:	2.3 kg	2.5kg	2.3 kg	2.3 kg	1.9 kg	2.4kg
Maximum weight user	136 kg	136 kg	136 kg	136 kg	110 kg	120 kg

**SYMBOLE:**

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

**WARRANTY CARD**  
The card must be accompanied by proof of purchase

<b>Sale date</b>	<b>Seller's stamp and signature</b>	
<b>Purchase invoice number</b>		
<b>LOT No</b>	<b>Guarantee period</b>	.....
<b>Name and catalog number product</b>	<b>Is the product introduced into the public reimbursement system?</b>	* delete inappropriate <b>YES NO*</b>
	<b>Order registration number</b> .....	

1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a customary 12-month warranty for the products covered by this document. If the product is introduced into the public refund system, the manufacturer provides a 30-month warranty period. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, plastic and rubber elements (i.e. abrasion of rubber caps, wheels for models AR-003, AR-008, AR-009, plastic handles).
6. The warranty does not cover if the walker is used contrary to its intended use or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
8. **The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.

The advertised product must be in proper hygienic condition.



**USER MANUAL  
WARRANTY CARD**

**ALUMINUM WALKERS**



Aluminum walker rolling-steady Model: AR-001



Aluminum walker rolling-steady Model: AR-002



Folding aluminum walker, two-wheeled Model: AR-003



Aluminum walker- foldable 3-function Model: AR-008



Aluminum walker with a ball foldable, two-wheeled Model: AR-009



Aluminum walker - rolling-steady making it easier to get up Model: AR-023

**ARMEDICAL Sp. z o.o.**  
al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska  
tel.fax: +48 32 261 71 36, +48 783 001 111  
www.armedical.pl, e-mail: biuro@armedical.pl





## USER MANUAL - ALUMINUM WALKERS

### Ladies and Gentlemen,

Thank you for purchasing our products, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will make it much easier for you to move around at home as well as during everyday walks. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

### Intended use of the product:

The walker is an auxiliary and rehabilitation equipment intended for disabled people with musculoskeletal disorders, problems with standing or having problems maintaining balance while walking. It is also used to maintain balance while learning to walk after an illness or injury.



### Contraindications:

Contraindications to upright positioning, injuries and dysfunctions of the upper limbs that make it impossible to support oneself on a walker, injuries of the lower limbs that make it impossible to maintain balance, motor coordination disorders, problems with the labyrinth, spinal instability.

### Usage data:

Maximum patient weight ensuring safe use of the walker:

**136 kg** - (model: AR-001, AR-002, AR-003, AR-008), **110kg** - (model AR-009), **120kg** - (model AR-023)



### Notes and warnings:

To avoid an accident, please:

1. Check whether all aluminum tubes are straight and not broken (e.g. during transport)
2. Check whether all legs of the walker are set at the same height
3. Check whether there are rubber pads on all legs and whether they are all installed correctly
4. Check whether the wheels are securely screwed (applies to models AR-003, AR-008, AR-009)
5. Check whether all rivets permanently connect the walker elements
6. Do not unfold the walker in places where it may be slippery (spilled liquid, ice)



### To avoid injuries:

1. The walker should be fully unfolded (the locking latch in the crossbar must engage in the hole in the crossbar).
2. It is necessary to check the stability of the walker each time before use.
3. All legs of the walker must be set at the same height.
4. Use the walker only on flat, hard ground. Do not use on stairs!
5. Be especially careful on wet or slippery surfaces.
6. When moving, be especially careful about lying rugs, pieces of clothing, toys, etc., as well as lying animals, and uneven, cracked surfaces.
7. Walkers cannot be exposed to extreme weather conditions for a long time (e.g. very severe frost or direct strong sunlight).
8. Do not overload the walker or exceed the maximum load allowed by the manufacturer.
9. Consult your doctor before use.

### Maintenance method:

The walker should be cleaned regularly using only water and mild detergents. The balcony should be wiped/dried with a soft cloth. Do not use abrasive products! The elements, i.e. (wheels, handles, rubber leg caps) are consumable elements and must be replaced with new ones if excessive wear is found. The above-mentioned items are not subject to warranty. Any looseness in screw connections is the responsibility of the user and does not constitute grounds for complaint.

### Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the walker yourself is FORBIDDEN!!! This may result in an accident.

#### I. The balcony is not stable.

- a. Are all the legs of the walker at the same height? If not, set them correctly.
- b. Do all legs have rubber caps? If not, put them on.
- c. Are all rubber caps installed correctly? If not, press them in firmly.
- d. Is the walker fully unfolded? If not, press the locking latch on the crossbar into the hole in the crossbar.
- e. Are the wheels of the walker screwed on securely? If not, please contact your point of purchase. Applies to model AR-003, AR-008, AR-009
- f. Are all the rivets connecting the walker elements solid? If not, please contact your point of purchase. DO NOT USE A BALCONY!!!

### Incidents:

Every **serious** incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

### Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

### Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

## COMPONENTS/UTILITY ELEMENTS



### Preparation for use

1. Remove the walker from the cardboard or foil packaging.
2. First, check the technical condition of the walker in accordance with point **Notes and warnings**.
3. Grasp the plastic handles (1) with both hands and move the handrails outwards until the walker is fully unfolded. Locking latch on the crossbar (4) must jump into the hole in the crossbar (when folding the walker, press the lock (4) located in the middle of the crossbar with your hand and then move one of the handrails inwards. This will allow you to unlock the latch and fold the second handrail, thus folding the walker. Unfolding the model AR-009 - grab the frame with wheels with one hand, the frame with feet with the other and spread it as far as possible. Then slide the transverse frame to the "limiters". Folding the AR-009 model - pull the ball up until it is completely folded.
4. Adjust the height of the walker to the user's height by pressing the head of the latch (5) located in the leg, which will release the lock. Set the required height by sliding or sliding. The latch head must engage in the hole at the desired height! Repeat the procedure for all legs of the walker. The latch head on all legs must be at the same height.
5. Walking walker (applies to models AR-001, AR-002, AR-008, AR-023). Remove the locking plugs (2) located in the hole from position (3) and then insert them into the frame holes to position (2). Place your hands on the handles (1) simultaneously. Lift the right or left part of the walker slightly and move it in the desired direction, add the leg. Perform this activity alternately in a "walking" motion.

**Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.**

### End of use:

1. Fold the walker by pressing the locking latch (4) or (pull the ball towards you - model AR-009) and then move one of the handrails inwards, this will release the lock and thus enable you to fold the walker.

### Cleaning and storage:

1. Place the walker in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.

**THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT**