

Technical data:

PARAMETER:	Model AR-004
Frame:	Steel
Handle height adjustment:	82-98 cm
Wheels:	Diameter 8" (20 cm), solid, maintenance-free
Weight:	8.3kg
Maximum user weight	110 kg
Standard equipment:	Brake with parking function, removable basket with cover, shopping bag with zipper

SYMBOLE:								
								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature	
Purchase invoice number		
LOT No	Guarantee period	
Three-wheeled walker with bag and basket Model: AR-004	Is the product introduced into the public reimbursement system?	* delete inappropriate YES NO*
	Order registration number	

- ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a customary 12-month warranty for the products covered by this document. If the product is introduced into the public refund system, the manufacturer provides a 30-month warranty period. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
- During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
- ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
- If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
- The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, plastic and rubber elements (i.e. abrasion of wheels, plastic/foam handles)
- The warranty does not cover if the support is used contrary to its intended purpose or in accordance with the user manual, or if the user exceeds the maximum permissible load specified by the manufacturer (applies to the structure of the walker, tray and shopping bag).
- The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
- The warranty is valid only with the proof of purchase and this warranty card.**
- Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded. **The advertised product must be in proper hygienic condition.**



USER MANUAL
WARRANTY CARD

THREE-WHEEL WALKER
WITH BAG AND BASKET

Model: AR-004



ARMEDICAL Sp. z o.o.

al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - THREE-WHEEL WALKER WITH BAG AND BASKET

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will make it much easier for you to move around at home as well as during everyday walks. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The walker is an auxiliary and rehabilitation equipment intended for disabled people with musculoskeletal disorders, problems with standing or having problems maintaining balance while walking. It is also used to maintain balance while learning to walk after an illness or injury.



Contraindications:

Contraindications to vertical positioning, injuries and dysfunctions of the upper limbs that make it impossible to support oneself on a walker, motor coordination disorders, problems with the labyrinth, spinal instability.

Usage data:

Maximum patient weight ensuring safe use of the walker – **110 kg** Maximum permissible load for basket with lid and shopping bag – **5 kg**



Notes and warnings:

To avoid an accident, please:

1. Check whether all frame elements (8) are straight and not broken (e.g. during transport)
2. Check whether both handles of the walker (1) are set at the same height
3. Check whether the screw clamps (5) are tightly tightened, preventing removal of the push handles (1)
4. Check if the walker has all three wheels (9) and (10)
5. Check whether the wheels (9) and (10) are securely fastened
6. Check whether all elements of the walker frame are stably connected to each other (8)
7. Check whether the crossbeam is fully unfolded (7) with the locking screw screwed in
8. Do not unfold/use the walker in places where it may be slippery (spilled liquid, ice)



To avoid injuries:

1. The walker should be fully unfolded (the screw in the crossbeam (7) must be securely tightened)
2. It is necessary to check the stability of the walker each time before use
3. The handles (push handles) (1) must be set at the same height
4. Use the walker only on flat, hard ground. Do not use on stairs!
5. Be especially careful on wet, slippery surfaces. Use the walker only when walking forward.
6. When moving, be especially careful about lying rugs, clothing items, toys, etc., as well as lying animals, uneven, cracked surfaces, uneven sidewalks, curbs, etc.
7. The walker must be held with both hands each time! Hands hold the handles stably (1), you can quickly pull the brake handles (2).
8. The walker cannot be exposed to extreme weather conditions for a long time (e.g. very severe frost or direct strong sunlight).
9. Do not overload the walker or exceed the maximum load allowed by the manufacturer.
10. Do not exceed the maximum permissible load of the basket with cover (3)
11. Do not exceed the maximum load capacity of the zippered bag (4)
12. Consult your doctor before use.

Maintenance method:

The walker should be cleaned regularly using only water and mild detergents. The balcony should be wiped/dried with a soft cloth. Do not use abrasive products! Brake cables and elements (wheels, handles) are wearing parts and must be replaced with new ones if excessive wear is found. The above-mentioned items are not subject to warranty. Any looseness in screw connections and brake adjustments are the responsibility of the user and do not constitute grounds for complaint.

Maintenance schedule:

Every week Check brake for failure, check all wheels for loose bolts, nuts and axles.

Periodically Check the grips for wear, check the frame bolts are tight, check the front and rear axle bolts are tight, check the "fatigue" components for wear or cracks.

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the walker yourself is **FORBIDDEN!!!** This may result in an accident.

I. The walker is unstable.

- a. Are both walker handles (1) at the same height? If not, set them correctly.
- b. Are the screw clamps (5) tightened securely? If not, tighten them.
- c. Is the crossmember (7) fully unfolded with the screw securely attached? If not, unfold it and tighten it with the locking screw.
- d. Are all wheels tightened securely? If not, tighten them securely.
- e. Are all elements of the walker frame (5) securely connected to each other? If not, please contact your point of purchase. **DO NOT USE A WALKER!!!**

Incidents

Everyone **serious** an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

COMPONENTS/UTILITY ELEMENTS



1. Handles, push handles
2. Handbrake with parking lock
3. Basket with cover
4. Bag closed with a zipper
5. Screw clamp
6. Brake cables
7. Crossbar folding/unfolding
8. Steel frame
9. Swivel front wheel
10. Rear wheels with pressure brake
11. Handle tubes

Preparation for use

1. Remove the walker from the cardboard or foil packaging.
2. First, check the technical condition of the walker in accordance with point **Notes and warnings**.
3. Insert the handles and push handles (1) into the holes in the frame (8). See the drawing above.
4. Grab the handles and push handles (1) with both hands and move them outwards until the walker is fully unfolded. Tighten the screw in the crossmember (7) securely.
5. Adjust the height of the walker and push handles (1) to the user's height. To do this, unscrew the screw clamp (5) located in the frame (8), removing the screw completely. Set the required height by sliding or sliding. Re-insert the screw into the existing hole in the frame, tighten and tighten with the screw clamp (5). Repeat the operation for both handles of the walker. The push handles (1) must be at the same height.
6. After disassembly, it is essential to check the correct operation of the brakes. Place your hands on the handles (1), pull the brake handles up (2) with your fingers and then check whether the clamps lock the rear wheels. Additionally, after the test, the brake handles (2) should be pushed down as far as possible with your fingers until the latch "clicks". The brake should be in the parking position and then check whether the wheels are locked by pushing the walker slightly forward.
7. Place the basket with cover (3) on the existing hooks in the front part of the walker. See the drawing above.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Lift the basket with the cover upwards (3), thereby removing it from the mounting brackets.
2. Fold the walker by folding the crossbar after loosening the locking screw, then move the handles (1) inwards, the walker will be folded.

Cleaning and storage:

1. Place the walker in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT