

Technical data:

PARAMETER:	AR-005	AR-006	AR-007	AR-020	AR-021	AR-022
Frame:	steel	aluminum	aluminum	steel	aluminum and steel	aluminum
Frame color:	blue	burgundy, silver, blue	silver nickel	blue	graphite	claret
Type/model:	folding	folding	folding	folding	folding	folding
Max. total width:	61 cm	59 cm	62 cm	49 cm	59 cm	60 cm
Seat height:	61 cm	52-60 cm	56 cm	52 cm	49 cm	54 cm
Height adjustment hands:	75-98 cm	94-101 cm	81-94 cm	75.5-92 cm	79-88 cm	79-91 cm
Product weight:	9.5kg	6kg	6.1kg	4.5kg	4.8 kg	8.2 kg
Maximum load:	136 kg	136 kg	136 kg	136 kg	136 kg	136 kg
Equipment:	basket, tray	mesh container	bag	-	-	basket

SYMBOLE:

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD
The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature	
Purchase invoice number		
LOT No	Guarantee period
Product name and catalog number	Is the product introduced into the public reimbursement system?	* delete inappropriate YES NO*
	Order registration number	
	

- ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a customary 12-month warranty for the products covered by this document. If the product is introduced into the public refund system, the manufacturer provides a 30-month warranty period. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
- During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
- ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
- If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
- The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, plastic and rubber elements (i.e. abrasion of wheels, plastic/foam handles)
- The warranty does not cover if the support is used contrary to its intended purpose or in accordance with the user manual, or if the user exceeds the maximum permissible load specified by the manufacturer (applies to the structure of the support, trays and shopping bags).
- The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
- The warranty is valid only with the proof of purchase and this warranty card.**
- Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded. **The advertised product must be in proper hygienic condition.**



**USER MANUAL
WARRANTY CARD**

REHABILITATION ROLLATORS

AR-005 Rehabilitation rollator
4-wheel, steel, CLASSIC



AR-006 Rehabilitation rollator
4-wheel, aluminum, PRESTIGE



AR-007 Rehabilitation rollator
4-wheel, aluminum, DYNAMIC



AR-022 Rehabilitation rollator
4-wheel, aluminum, COMFORT



AR-021 Rehabilitation rollator
two-wheeled, DELUXE



AR-020 Rehabilitation rollator
two-wheeled, STANDARD

ARMEDICAL Sp. z o.o.
al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - REHABILITATION ROLLATORS

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will make it much easier for you to move around at home as well as during everyday walks. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The rehabilitation support is an auxiliary and rehabilitation equipment intended for disabled people with musculoskeletal disorders, problems with standing or having problems maintaining balance while walking. It is also used to maintain balance while learning to walk after an illness or injury.



Contraindications:

Contraindications to vertical positioning, injuries and dysfunctions of the upper limbs that make it impossible to support oneself on a support, impaired motor coordination, problems with the labyrinth, spinal instability.

Usage data:

Maximum patient weight ensuring safe use of the rollator -136 kg

Maximum permissible load of shopping containers (basket, bag) -5kg



Notes and warnings:

To avoid an accident, please:

1. Check whether all frame elements (7) are straight, not broken or damaged (e.g. during transport)
2. Check whether both handles/handles for pushing the support (1) are set at the same height
3. Check whether the handles/handles (1) of the supports are firmly "clamped" and screwed, making it impossible to remove them
4. Check whether the support has all four wheels (5) and (8) - AR-020 - 2 wheels.
5. Check whether the wheels (5) and (8) are securely fastened
6. Check whether all elements of the support frame are stably connected to each other (7)
7. Check whether the support is fully extended
8. Do not unfold the support in places where it may be slippery, e.g. spilled liquid or ice
9. **Do not treat the support as a wheelchair - applies to the AR-022 model**



To avoid injuries:

1. The support must be fully unfolded (when properly unfolded, it must look exactly as in the photo)
2. It is necessary to check the stability of the support each time before use
3. The handles (push handles) (1) must be set at the same height
4. Use the support only on a flat, hard surface. Do not use on stairs!
5. Be especially careful on wet, slippery surfaces.
6. When moving, be especially careful about lying rugs, clothing items, toys, etc., as well as lying animals, uneven, cracked surfaces, uneven sidewalks, curbs, etc.
7. The support must always be held with both hands! Hands hold the handles stably (1), you can quickly pull the brake handles (2). (does not apply to AR-020, AR-021)
8. The support cannot be exposed to extreme weather conditions for a long time (e.g. very severe frost or direct sunlight).

9. Do not overload the support, exceed the maximum load allowed by the manufacturer.
10. Do not exceed the maximum permissible load of the basket/bag (3)
11. Consult your doctor before use.

12. Model AR-022 - NEVER STAND ON THE FOOTREST. RISK OF ACCIDENT!

Maintenance method:

The support should be cleaned regularly using only water and mild detergents. The support should be wiped/dried with a soft cloth. Do not use abrasive products! Brake cables and elements (wheels, handles) are wearing parts and must be replaced with new ones if excessive wear is found. The above-mentioned items are not subject to warranty. Any looseness in screw connections and brake adjustments are the responsibility of the user and do not constitute grounds for complaint.

Maintenance schedule:

Every week Check brake for failure, check all wheels for loose bolts, nuts and axles

Periodically Check the grips for wear, check the frame bolts are tight, check the front and rear axle bolts are tight, check the "fatigue" components for wear or cracks.

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the support yourself is FORBIDDEN!!! This may result in an accident.

I. The support is not stable.

- a. Are both support handles (1) at the same height? If not, set them correctly.
- b. Are the clamps/screws (6) firmly clamped/tightened? If not, tighten/tighten them.
- c. Is the kickstand fully extended? If not, unfold it and check its stability.
- d. Are all wheels (5) and (8) tightened securely? If not, tighten them securely.
- e. Are all components of the support frame (7) securely connected to each other? If not, please contact your point of purchase. DO NOT USE A STAND!!!

Incidents:

Everyone serious an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

COMPONENTS/UTILITY ELEMENTS



ATTENTION!
NEVER STAND ON THE FOOTREST (9)
RISK OF ACCIDENT!

1. Handles / Push handles
2. Brake handles with parking function
3. Basket/shopping bag
4. Seat
5. Wheels with brake
6. Clamps / Screws / Handle latches
7. Frame
8. Swivel wheels (fixed AR-020, AR-021)
9. Footstool
10. Backrest
11. Rubber feet

Preparation for use

1. Remove the support from the cardboard or foil packaging.
2. First, check the technical condition of the support in accordance with point Notes and warnings.
3. Insert the handles and push handles (1) into the holes in the frame (7). See the drawing above.
4. Clamp/screw the handles and push handles (1) to the frame (7) so that they cannot be removed
5. Adjust the height of the support and push handles (1) to the user's height. To do this, unscrew/remove the clamp (AR-005), the screw (AR-006, AR-020, AR-021, AR-022), the latch (AR-007) (6) located in the frame (7). Set the required height by sliding or sliding. Re-insert the screw/latch into the existing hole in the frame, tighten/clamp. Repeat the operation for both support handles. Push the handles (1) They must be at the same height.
6. After disassembly, it is essential to check the correct operation of the brakes (does not apply to AR-020, AR-021). Place your hands on the handles (1), pull the brake handles up (2) with your fingers and then check whether the clamps lock the wheels with brakes (5). Additionally, after the test, the brake handles (2) should be pushed down as far as possible with your fingers until the latch "clicks". The brake should be in the parking position and then check whether the wheels are locked by pushing the walker slightly forward.
7. Install the basket, tray (AR-005, AR-022), bag (AR-007) (3). See the drawing above.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Fold the rollator by pushing the front wheels and rear wheels/legs together (AR-005, AR-006, AR-020, AR-021, AR-022).
2. In the AR-006, AR-020, AR-021, AR-022 models, the folding handle is located under the seat. Fold the support (AR-007) by pushing the right and left wheels together by pulling the handle on the seat.

Cleaning and storage:

1. Place the support in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT