

Technical data:

PARAMETER:	Model AR-110 / AR-110BK	Model AR-115 / AR-115BK
Outer width:	36.5 cm	36.5 cm
External length:	40 cm	40 cm
Inside opening width:	20.5 cm	20.5 cm
Length of hole inside:	26 cm	26 cm
Raise height:	10 cm	15 cm
Maximum user weight:	180 kg	180 kg

SYMBOLE:

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature
Purchase invoice number	
Batch No	

1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a 12-month warranty for the products covered by this document. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only within the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, i.e. scratches, scuffs, abrasions on plastic elements as well as color changes resulting from storage in a sunny place.
6. The warranty does not cover if the overlay is used contrary to its intended use or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
- 8. The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.

The advertised product must be in proper hygienic condition.




**USER MANUAL
WARRANTY CARD**

RAISED TOILET SEAT WITH FLAP

Models: AR-110, AR-115

RAISED TOILET SEAT

Models: AR-110BK, AR-115BK

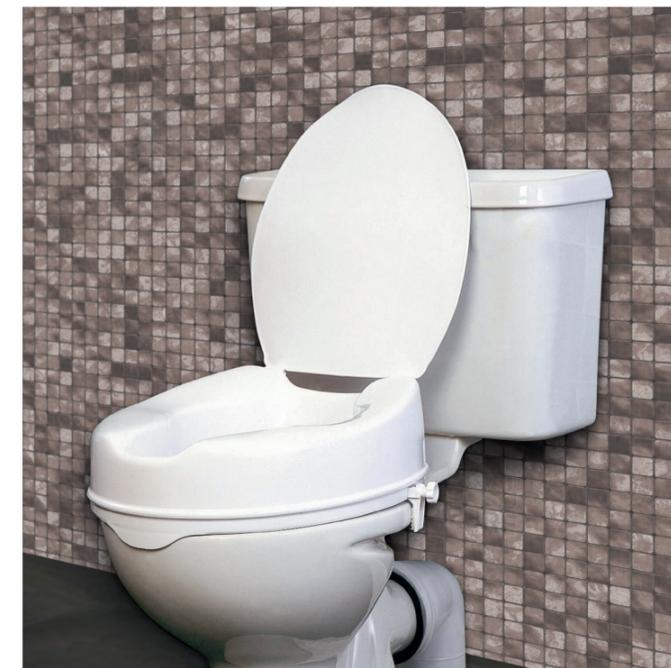

ARMEDICAL Sp. z o.o.
 al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
 tel.fax: +48 32 261 71 36, +48 783 001 111
 www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - RAISED TOILET SEAT

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will significantly facilitate your daily toileting activities. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The toilet seat is intended for disabled people with musculoskeletal disorders, especially those who have trouble getting up and sitting down. Elderly people as well as people with arthritis, after hip surgery, who have problems using a standard height toilet bowl. Makes everyday toileting easier and easier. It has a modern mounting system that allows for quick, easy and stable mounting on the toilet. The overlay is made of profiled material with a special coating that is easy to clean and disinfect. The size standard used allows it to fit most toilets. Adjustable side clamps ensure stable positioning of the cover on the existing toilet. Appropriate profile, "indentation" in the front part ensures comfort of use for both women and men. Thanks to the cover (applies to models: AR-110, AR-115), it is possible to close the toilet when it is not in use. The ergonomic shape of the product ensures comfort of use.

⚠️ Contraindications:

Contraindications to a sitting position, lack of trunk stabilization. Movement coordination and balance disorders, labyrinth problems, spinal instability.

Usage data:

Maximum patient weight ensuring safe use of the toilet seat – **180 kg**

⚠️ Notes and warnings:

To avoid an accident, please:

1. Check whether all elements are intact and not broken (e.g. during transport)
2. Checking whether all plastic elements have no damage, e.g. cracks
3. Check whether the side clamps and the screw are not damaged
4. Check whether there are rubber pads on the side clamps
5. Do not put the cover on a wet toilet

⚠️ To avoid injuries:

1. The cover should adhere with its entire bottom surface to the existing toilet
2. It is necessary to check the technical condition and stability of the product each time before use (checking all fastening elements)
3. Use only after the cover is firmly installed (tightening the pressing elements with a screw)
4. Be especially careful on wet or slippery surfaces.
5. It is forbidden to sit on the pad abruptly (including jumping)
6. It is forbidden to stand with shoes on the cover, flap (applies to AR-110, AR-115) or any element of the product.
7. Do not overload the pad, exceed the maximum load specified by the manufacturer.
8. You should only sit on the pad when the flap is open (raised) (applies to AR-110, AR-115)

Maintenance method:

The overlay should be cleaned regularly using only water and mild detergents certified by the National Institute of Hygiene and disinfectants approved for use on plastics. Then dry thoroughly with a soft cloth. The use of strong detergents, gasoline and solvents is prohibited. Storage in rooms with high humidity, e.g. (basement), combined with lack of thorough product maintenance (hygiene) will result in the loss of the functional and aesthetic parameters of the overlay and thus the loss of the warranty. Do not use abrasive products! Do not store in sunny places. **Let's remember! The product is a toilet cover, so hygiene rules must be particularly observed.**

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing yourself (gluing, screwing) is FORBIDDEN!!! This may result in an accident.

I. The toilet seat is not stable.

- a. Are the pressure elements screwed on securely? If not, screw them on
- b. Do the pressure elements have rubber caps? If not, do not use the product.
- c. Does the entire surface of the cover adhere to the toilet? If not, set it correctly.
- d. Is any part cracked/broken? If so, please contact your point of purchase. DO NOT USE A TOILET COVER!

Incidents:

Everyoneserious an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

COMPONENTS/UTILITY ELEMENTS



Preparation for use

1. Remove the toilet seat from the cardboard or foil packaging.
2. First, check the technical condition of the product in accordance with point **Notes and warnings**.
3. Grasp with both hands and place on the existing toilet seat (with the lid open). Check whether it adheres to the entire bottom surface.
4. Adjust the width of the side clamps to the existing toilet and then tighten them tightly. After installation, check the stability of the toilet cover by moving it sideways.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Dismantle the toilet cover by unscrewing the screws of the side clamps, then remove the cover from the existing toilet by lifting it up with both hands.

Cleaning and storage:

1. Place the toilet seat in a place where it will not be exposed to mechanical damage or sunlight.
2. In case of dirt, follow the "Maintenance method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT