

Technical data:

PARAMETER:	AR-210
Frame:	aluminum
Frame color:	silver
Type/model	constant
Seat width:	41 cm
Seat depth:	21.5 cm
Height adjustment	21cm-26cm
Product weight:	1.7 kg
Maximum load:	160 kg

SYMBOLS:								
								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

The card must be accompanied by proof of purchase

Sale date	Seller's stamp and signature
Purchase invoice number	
Product no. / LOT no	

1. 1. ARMEDICAL Sp. z o. o. 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a 12-month warranty for the products covered by this document. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, i.e. scratches, scratches on polyethylene and rubber elements (i.e. abrasion of rubber caps (4).
6. The warranty does not cover if the footrest/shower stool is used contrary to its intended use or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer.
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
- 8. The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.
- The advertised product must be in proper hygienic condition.**



USER MANUAL
WARRANTY CARD

FOOTREST – SHOWER STOOL

MODEL: AR-210



ARMEDICAL Sp. z o.o.

al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - FOOTREST - SHOWER STOOL

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will significantly facilitate your daily toileting activities. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

Footrest - shower stool as an auxiliary equipment is intended for disabled people with disorders of the musculoskeletal system, elderly people as well as people for whom, for health reasons, the standard use of a bathtub or a shower cabin is a problem. They facilitate and help with everyday toilet activities, e.g. at the sink, in the bathtub or in the shower. Used as a step, it makes it easier to enter the bathtub for people who have problems with it due to limited mobility.



Contraindications:

Contraindications to a sitting position, lack of trunk stabilization. Disorders of motor coordination and balance, labyrinth problems, spinal instability, instability of the lower limbs making it impossible to stand on a footrest. Intellectually limited people should not use the product without the supervision of a guardian.

Usage data:

Maximum patient weight ensuring safe use of the product –**160 kg**



Notes and warnings:

To avoid an accident, please:

1. Check whether all tubes are straight and not broken (e.g. during transport), and whether all product components are included in the packaging.
2. Check whether the polyethylene seat/step has no damage, e.g. cracks.
3. Check whether all legs of the footrest/stool are set at the same height.
4. Check whether all legs have anti-slip rubber pads and whether they are all correctly installed.



To avoid injuries:

1. The polyethylene seat/step must be firmly and permanently attached (using the supplied screws (5)) to the aluminum frame (2)
2. It is necessary to check the technical condition and stability of the product each time before use (checking all fastening elements, product frame tubes and rubber caps)
3. All legs of the shower footrest/stool must be set at the same height.
4. Use only on a flat, hard surface that prevents the footrest/stool from moving.
5. Be especially careful on wet or slippery surfaces despite the non-slip caps.
6. It is forbidden to suddenly step on or sit on the footrest - stool (including jumping).
7. It is prohibited to move the footrest/stool with the user.
8. Do not overload the product or exceed the maximum load specified by the Manufacturer.
9. **Be especially careful when standing on the footrest/stool after toileting (e.g. leaving the bathtub). Due to residues of soap or other cleaning agents, the step/seat surface may be very slippery. VIOLENT STANDING MAY RESULT IN AN ACCIDENT!**

Maintenance method:

Each time after toileting, the product should be thoroughly rinsed with clean water (do not leave any soap, shampoo, bath gel, etc. on the product) and then dried thoroughly with a delicate cloth. It is prohibited to use strong detergents, gasoline or solvents for cleaning. Storing in rooms with high humidity, e.g. (basement), combined with the lack of thorough maintenance of the product (hygiene, drying) will result in the loss of the functional and aesthetic parameters of the footrest-stool and thus the loss of the warranty. Do not use abrasive products! Anti-slip rubber elements (leg caps) (4) are wearing parts and must be regularly replaced with new ones.

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the product yourself is FORBIDDEN!!! This may result in an accident.

I. Footrest - the shower stool is not stable.

- a. Are all the legs of the footrest/stool at the same height? If not, set them correctly.
- b. Do all legs have rubber caps (4)? If not, put them on.
- c. Are all rubber caps (4) installed correctly? If not, press them in firmly.
- d. Does the product have a polyethylene element firmly screwed to the frame (2) (with included screws (5))? If not, retighten all screws firmly. All product elements should be assembled/assembled in such a way that, after completing the operation, they constitute an exact reproduction of the product photo on page 1.
- e. Even though all screws (5) are firmly tightened, is the product still unstable? If not, please contact your point of purchase. **DO NOT USE A FOOTREST - A SHOWER STOOL!!!**

Incidents:

Everyone's **serious** an incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.


Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

COMPONENTS/UTILITY ELEMENTS

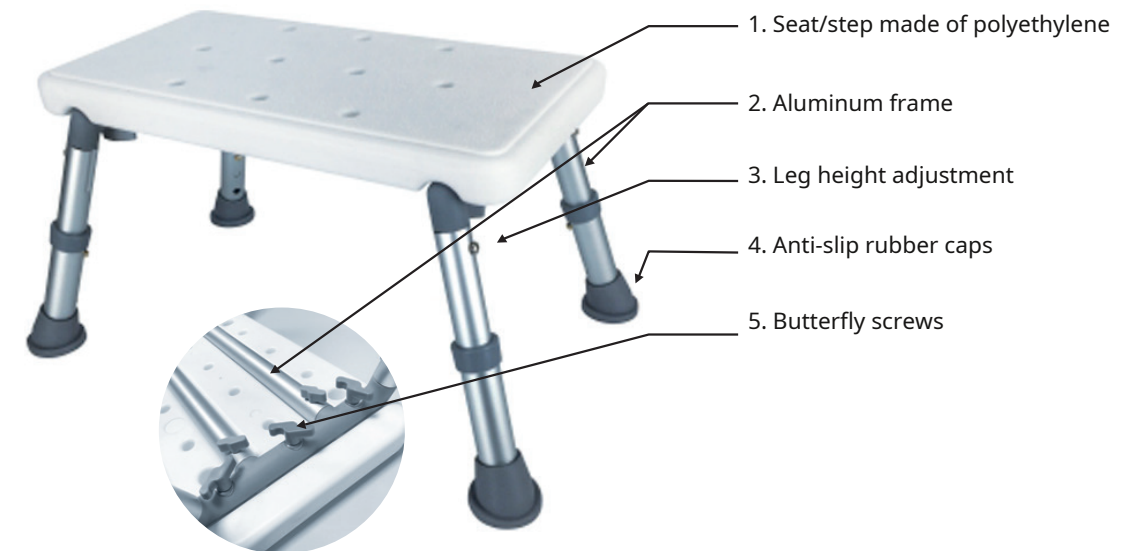

Preparation for use

1. Remove the footrest - shower stool from the cardboard or foil packaging. The product removed from the packaging requires assembly before first use.
2. First, check the technical condition of the product in accordance with point **Notes and warnings**.
3. Place the seat/step (1) on the floor or hard surface so that the bottom part is visible. Grasp the aluminum frame elements (2) and place them with the rubber caps facing up (4) so that the holes in the frame elements and the footrest seat line up with each other. Screw in all included screws into existing holes.
ATTENTION! All screws must be tightened tightly to ensure safe use of the product.
4. Adjust the height of the legs of the footrest - shower stool to the user's height. To do this, press the head of the latch (3) located in the leg, which will release the lock. Set the required height by extending or sliding the tube. The latch head must engage in the hole at the desired height! Repeat the operation for all legs. The latch head on all legs must be at the same height.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

End of use:

1. Place the footrest - shower stool in a place where it will not be exposed to mechanical damage; the product can also remain "in" or "near" the bathtub if it does not constitute a tripping hazard when walking.

Cleaning and storage:

1. In case of dirt, follow the "Maintenance method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT