

TECHNICAL PARAMETERS:	Model AR-030
Frame: aluminum - foldable	Total dimensions width x length: 65x61cm
Handle height adjustment: 81-95 (every 2.5 cm)	Folded width: 20cm
Wheels: Diameter 8" (20cm), front swivel wheel	full, maintenance-free, anti-slip EVA
Libra: 4.60kg	Maximum user weight: 110 kg Maximum load on a shopping bag: 5kg
Standard equipment:	Brake with parking function, nylon shopping bag, zippered - size: 45x35x30cm

SYMBOLE:								
								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD

Sale date	Seller's stamp and signature	
Purchase invoice number		
LOT No	Guarantee period
	Is the product introduced into the public reimbursement system?	* delete inappropriate YES NO*
	Order registration number	
.....		

1. ARMEDICAL Sp. z o. o., 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a customary 12-month warranty for the products covered by this document. If the product is introduced into the public refund system, the manufacturer provides a 30-month warranty period. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
4. If it is found that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
5. The warranty does not cover damage caused by mechanical factors. The warranty does not cover the subject of sale in terms of wear and tear elements during normal use, plastic and rubber elements (i.e. abrasion of wheels, plastic handles)
6. The warranty does not cover if the walker is used contrary to its intended purpose or in accordance with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer (applies to the structure of the walker, bag closed with a zipper).
7. The warranty does not cover defects resulting from unauthorized repairs, modifications or design changes to the product made by the User or other unauthorized persons.
- 8. The warranty is valid only with the proof of purchase and this warranty card.**
9. Complaints should be submitted only in the form of a written complaint protocol specifying the type of defects found and the circumstances in which the defect was discovered. In case of failure to comply with the conditions for filing complaints specified in this manual, the guarantor's liability is excluded.
The advertised product must be in proper hygienic condition.



**USER MANUAL
WARRANTY CARD**

3-WHEELS WALKER WITH SEAT

Model: AR-030 SUPER LIGHT



ARMEDICAL Sp. z o.o.
 al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
 tel.fax: +48 32 261 71 36, +48 783 001 111
 www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL

ALUMINUM 3-WHEEL WALKER WITH A SEAT

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will make it much easier for you to move around at home as well as during everyday walks. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Intended use of the product:

The walker is an auxiliary and rehabilitation equipment intended for disabled people with musculoskeletal disorders, problems with standing or having problems maintaining balance while walking. It is also used to maintain balance while learning to walk after an illness or injury.



Contraindications:

Contraindications to vertical positioning, injuries and dysfunctions of the upper limbs that make it impossible to support oneself on a walker, coordination disorders, labyrinth problems, spinal instability.



Notes and warnings:

1. Check whether all frame elements (8) are straight and not broken (e.g. during transport)
2. Check whether both walker handles (1) are set at the same height and the screws are securely tightened
3. Check if the walker has all three wheels (13) and (14)
4. Check whether the wheels are securely screwed/attached (13) and (14)
5. Check whether all elements of the walker frame are stably connected to each other (8)
6. Check whether the latch (7) blocking the folding is completely blocked
7. Check whether the front wheel mounting bolt (10) is securely tightened



To avoid injuries:

1. The walker should be fully unfolded, the latch (7) under the seat (in the front part) must be completely locked
2. It is necessary to check the stability of the walker each time before use
3. The handles (push handles) (1) must be set at the same height
4. Use the walker only on flat, hard ground. Do not use on stairs!
5. Be especially careful on wet, slippery surfaces.
6. When moving, be especially careful about lying rugs, clothing items, toys, etc., as well as lying animals, uneven, cracked surfaces, uneven sidewalks, curbs, etc.
7. The walker must be held with both hands each time! Hands hold the handles stably (1), you can quickly pull the brake handles (2).
8. The walker cannot be exposed to extreme weather conditions for a long time (e.g. very severe frost or direct strong sunlight).
9. Do not overload the walker or exceed the maximum load allowed by the manufacturer.
10. Do not exceed the maximum load capacity of the zippered bag (9)
11. Consult your doctor before use.
12. Independent repairs, modifications, design changes or modifications to the product by the user are prohibited. RISK OF ACCIDENT. The manufacturer is not responsible for the consequences of any design changes or modifications made to the product by the user.

Alternatively, please consult your doctor, rehabilitator or point of sale for detailed instructions on appropriate use.

Maintenance method:

The balcony should be cleaned regularly using only water and mild detergents. The balcony should be wiped and dried with a soft cloth. Do not use abrasive products! Brake cables and rubber elements (wheels, handles) are wearing parts and must be replaced with new ones if excessive wear is detected. The above-mentioned items are not subject to warranty. Any looseness in screw connections and brake adjustments are the responsibility of the user and do not constitute grounds for complaint.

Maintenance schedule:

Every week Check brake for failure, check all wheels for loose bolts, nuts and axles.

Periodically Check the grips for wear, check the frame bolts for tightness, check for the front and rear axle bolts for tightness, check for "fatigue" components for wear or cracks.

Annually Once a year, send your walker to an authorized service center and perform a full inspection, cleaning, service and lubrication of all moving parts.

Assembly instructions - preparation for use


Step 1: open and place the frame on the floor.

Step 2: insert the 2 push handle tubes into the frame and secure with 2 knob screws.

Step 3: insert the rear wheels into the frame. Before use, make sure that the latches engage in the holes in the frame and are securely locked. **Step 4:** install the front wheel and secure it with a screw. Make sure the screw head is properly seated in the hole.



COMPONENTS/UTILITY ELEMENTS

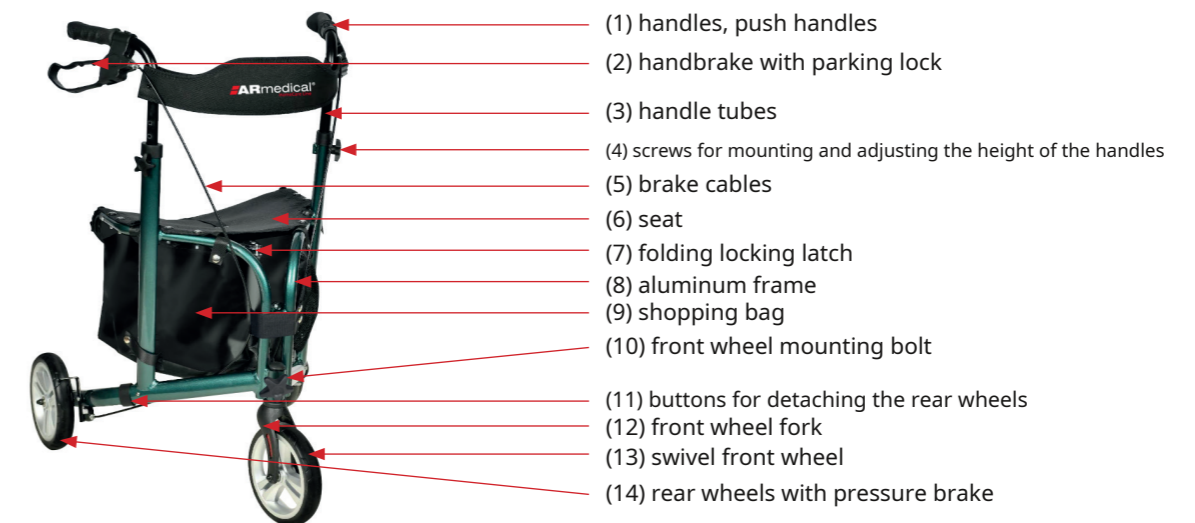

Troubleshooting:

If you encounter problems, please review this chapter. If this does not help, please contact the sales or service point. Repairing the walker yourself is FORBIDDEN!!! RISK OF ACCIDENT!!!

I. The balcony is unstable.

- a. Are both walker handles (1) at the same height? If not, set them correctly.
- b. Are the handle adjustment screws (4) tightened securely? If not, tighten them.
- c. Is the frame fully unfolded with the folding latch (7) engaged? If not, unfold it until the latch locks audibly.

d. Are all wheels (12,13) securely tightened/secured? If not, tighten/fix them securely.

e. Are all elements of the walker frame (8) securely connected to each other? If not, please contact your point of purchase. DO NOT USE A BALCONY!!!

Everyone's **serious incident** related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

End of use:

1. Fold the walker by pushing the rear wheels towards each other after pulling the handle in the seat and releasing the folding lock, then move the handles (1) inwards, the walker will be folded.

Cleaning and storage:

1. Place the walker in a place where it will not be exposed to mechanical damage.
2. In case of dirt, follow the "Maintenance method" section of this manual.