



USER MANUAL / WARRANTY CARD

Elbow ball - aluminum AR-010

Elbow ball - aluminum with a Softi handle. AR-010M

Elbow ball - aluminum with a movable AR-024 clamp

Elbow ball - aluminum with a movable clamp and a Pro-Soft soft handle. AR-024M

Elbow ball - aluminum with a movable clamp, ergonomic AR-025 handle

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will make it much easier for you to move around at home as well as during everyday walks. Before use, please read this user manual and strictly follow the safety regulations. Please keep this manual, which also serves as a warranty card.

Purpose and description of the product:

Elbow crutches are intended for people who need support while learning to walk after lower limb injuries, to maintain balance and to transfer body weight in traumatic and orthopedic diseases (fractures, dislocations, sprains). They are made of two durable and light aluminum tubes, plastic handles and a rubber (anti-slip) cap. Thanks to the use of an appropriately soft rubber mixture, the anti-slip caps perfectly absorb shocks. The painting technology guarantees the durability of the paint and its aesthetic appearance for a long time.



Contraindications:

Contraindications to vertical positioning, injuries and dysfunctions of the upper limbs that prevent support on a crutch, motor coordination disorders, labyrinth problems, spinal instability.

Usage data:

Maximum patient weight ensuring safe use of the elbow ball **-120 kg**



Notes and warnings:

To avoid an accident, please:

1. Check whether all elements are intact and not broken (e.g. during transport)
2. Checking whether all plastic elements have no damage, e.g. cracks
3. Check whether the ball has an anti-slip rubber cap installed
4. Check whether the head of the height adjustment latch is in the cut hole



To avoid injuries:

1. It is necessary to always check the technical condition and stability of the product before use (checking all elements)
2. Use only after adjusting the correct height to the user.
3. Be especially careful on wet or slippery surfaces.
4. Do not overload the product or exceed the maximum load specified by the manufacturer.
5. Use the ball only in the "locked" height adjustment latch position.
6. When moving, be especially careful about lying rugs, pieces of clothing, toys, etc., as well as lying animals, and uneven, cracked surfaces.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT

Maintenance method:The elbow ball should be cleaned regularly using only water and mild detergents certified by the National Institute of Hygiene. After doing so, the ball should be wiped/dried with a delicate cloth. Do not use abrasive products! The rubber elements (anti-slip rubber cap) are wear parts and must be regularly replaced with a new one.

Incidents:Any serious incident related to the device should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products

Utilization:A product that is no longer suitable for further use should be disposed of in accordance with local environmental regulations regarding product disposal.

Cessation of use of the product:In case of visible mechanical damage or material wear, stop using the product.

SYMBOLE:



Wyrób
medyczny



Zapoznaj
się z
instrukcją



Producent



Wyrób spełnia
wymagania
UE



Ostrzeżenie



Nr dato /
serii



Data
produkcji
m-c-rok



Nadajcie
się do
recyklingu



Selektywna
zbiórka
odpadów

1. ARMEDICAL Sp. z o. o., 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75, hereby grants an 18-month warranty for the elbow ball.
2. During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to its proper working condition within 14 days from the date of receipt of the complained goods.
3. ARMEDICAL Sp. z o. o. reserves the right to replace the product if it considers that the repair costs make it unprofitable.
4. If you find that the product is damaged, please contact the point of purchase.
5. Rights under the granted warranty will be exercised upon presentation of proof of purchase and warranty card
6. Products in good hygienic condition (clean) are accepted for complaints.
7. The warranty does not cover damage caused by mechanical factors or contact of the product with a hot surface (rubber and PVC elements). The warranty does not cover natural wear and tear of the product resulting from use or consumable elements (rubber caps).
8. The warranty does not cover if the product is used contrary to its intended purpose. The warranty does not cover damage resulting from modifications and design changes to the product made by the user or third parties.
9. In the event of an unjustified/rejected complaint, the costs of transport to the manufacturer are borne by the customer complaining about the goods.

Model/LOT number	
Sale date:	
Point of sale stamp and legible signature	



ARMEDICAL Sp. z o.o.

al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl

