

Technical data:

PARAMETER:	AR-005	AR-006	AR-007	AR-020	AR-021	AR-022	AR-032
Frame:	steel	aluminum	aluminum	steel	aluminum-steel	aluminum	aluminum
Frame color:	blue	burgundy, silver, blue	silver nickel	blue	graphite	claret	black
Type/model:	folding	folding	folding	folding	folding	folding	folding
Max total width:	61 cm	59 cm	62 cm	49 cm	59 cm	60cm	64 cm (with handle) for balls 67 cm)
Seat height:	61 cm	52-60cm	56 cm	52 cm	49 cm	54 cm	46-53cm
Height adjustment hands:	75-98cm	94-101cm	81-94cm	75.5-92cm	79-88cm	79-91cm	67-90cm
Product weight:	9.5kg	6kg	6.1 kg	4.5kg	4.8kg	8.2kg	6.8kg
Maximum load:	136kg	136kg	136kg	136kg	136kg	136kg	110kg
Equipment:	basket, tray	mesh container	bag	-	-	basket	bag

SYMBOLE:

								
Wyrób medyczny	Zapoznaj się z instrukcją	Producent	Wyrób spełnia wymagania UE	Ostrzeżenie	Nr dato / serii	Data produkcji m-c/rok	Nadaje się do recyklingu	Selektywna zbiórka odpadów

WARRANTY CARD
Proof of purchase must be attached to the card.

Sale date	Seller's stamp and signature	
Purchase invoice number		
LOT No.	Warranty period
Product name and catalog number	Is the product introduced into the public reimbursement system?	*delete where inappropriate YES / NO*
	Order reference number	
.....		

- ARMEDICAL Sp. z oo 42-520 Dąbrowa Górnicza, al. Zwycięstwa 75 hereby grants a customary 12-month warranty for the products covered by this document. In the event of introducing the product to the public funds refund system, the manufacturer grants a 30-month warranty period. The warranty period is counted from the date of purchase of the product by the User. The warranty is valid only in the territory of the Republic of Poland.
- During the warranty period, ARMEDICAL Sp. z o. o. undertakes to make the necessary repairs in order to restore the product to proper functioning within 14 days from the date of receipt of the complained goods.
- ARMEDICAL Sp. z o. o. reserves the right to replace the product with a defect-free one if it considers that the repair costs make it unprofitable.
- If you notice that the product is not functioning properly, please immediately contact the point of purchase or send it to the manufacturer's address.
- The warranty does not cover damage caused by mechanical factors. The warranty does not cover the item sold in terms of wear and tear elements during normal use, plastic and rubber elements (i.e. abrasion of wheels, plastic/foam handles)
- The warranty does not cover the use of the support in a manner inconsistent with its intended use or in a manner inconsistent with the operating instructions, or if the user exceeds the maximum permissible load specified by the manufacturer (applies to the structure of the support, trays and shopping bags)
- The warranty does not cover defects resulting from unauthorized repairs, alterations or structural changes to the product performed by the User or other unauthorized persons.
- The warranty is valid only with proof of purchase and this warranty card.**
- Complaints should be submitted exclusively in the form of a written complaint protocol specifying the type of defects found together with the circumstances in which the defect was discovered. Failure to comply with the terms of submitting complaints specified in this instruction excludes the guarantor's liability. **The complained product must be in proper hygienic condition.**



**USER MANUAL
WARRANTY CARD**

REHABILITATION ROLLATORS

<p>AR-005 Rehabilitation rollator 4-wheel, steel, CLASSIC</p> 	<p>AR-006 Rehabilitation rollator 4-wheel, aluminum, PRESTIGE</p> 	<p>AR-007 Rehabilitation rollator 4-wheel, aluminum, DYNAMIC</p> 	<p>AR-022 Rehabilitation rollator 4-wheel, aluminum, COMFORT</p> 
<p>AR-021 Rehabilitation rollator two-wheeled, DELUXE</p> 	<p>AR-020 Rehabilitation rollator two-wheeled, STANDARD</p> 	<p>AR-032 Rehabilitation rollator 4-wheel, aluminum PRACTICAL</p> 	

ARMEDICAL Sp. z o.o.
al. Zwycięstwa 75, 42-520 Dąbrowa Górnicza, Polska
tel.fax: +48 32 261 71 36, +48 783 001 111
www.armedical.pl, e-mail: biuro@armedical.pl





USER MANUAL - REHABILITATION SUPPORTS

Ladies and Gentlemen,

Thank you for purchasing our product, we value the trust you have placed in us. We have made every effort to ensure that our equipment meets your expectations, we hope that it will be a product that will significantly facilitate your movement at home as well as during everyday walks. Before use, please read this user manual, it is absolutely necessary to follow the safety regulations. Please keep this manual, which is also a warranty card.

Product purpose:

A rehabilitation support is an auxiliary and rehabilitation device designed for disabled people with a musculoskeletal disorder, with problems with verticalization or having problems with maintaining balance while walking. It is also used to maintain balance while learning to walk after an illness or injury.

Contraindications:

Contraindications to verticalization, injuries and dysfunctions of the upper limbs that prevent leaning on a support, impaired motor coordination, labyrinth problems, spinal instability.

Usage data:

Maximum patient weight to ensure safe use of the support -**136kg**(model: AR-032 -**110kg**) Maximum permissible load of shopping containers (basket, bag)-**5kg**

Notes and warnings:

To avoid accidents, please:

1. Check if all frame elements (7) are straight, not broken, damaged (e.g. during transport)
2. Check whether both handles/pull handles of the support (1) are set at the same height.
3. Check whether the handles/grips (1) of the support are tightly "tightened", screwed, preventing them from being removed
4. Check if the stand has all four wheels (5) and (8) - AR-020 - 2 wheels.
5. Check if the wheels are securely fastened (5) and (8)
6. Checking whether all the elements of the support frame are firmly connected to each other (7)
7. Check if the support is fully extended
8. Do not use the support in places where it may be slippery, e.g. spilled liquid, ice.
9. **The support should not be treated as a wheelchair - applies to model AR-022**

To avoid injuries:

1. The stand must be fully unfolded (when properly unfolded, it must look exactly as in the picture)
2. It is necessary to check the stability of the support each time before use.
3. The handles (pull handles) (1) must be set at the same height
4. Only use the stand on a flat, solid surface. Do not use on stairs!
5. Use extreme caution on wet, slippery surfaces.
6. When moving around, pay special attention to lying rugs, pieces of clothing, toys, etc., as well as lying animals, uneven, cracked surfaces, uneven pavements, curbs, etc.
7. The support must always be held with both hands! Hands firmly gripping the handles (1), possibility of quickly pulling the brake handles (2). (does not apply to AR-020, AR-021)
8. The stand must not be exposed to long-term extreme weather conditions (e.g. very heavy frost or direct strong sunlight)
9. Do not overload the support, do not exceed the maximum load allowed by the manufacturer.
10. The maximum permissible load of the basket/bag (3) must not be exceeded
11. Consult your physician before use.

12. Model AR-022 - NEVER STAND ON THE FOOTREST. DO NOT TRANSPORT THE PATIENT AS IN A WHEELCHAIR. RISK OF ACCIDENT!

Maintenance method:

The stand should be cleaned regularly using only water and mild detergents. The stand should be wiped/dried with a soft cloth. Do not use abrasive agents! Brake cables, components (wheels, handles) are consumables and must be replaced with new ones if excessive wear is detected. The above-mentioned components are not covered by the warranty. Any play in screw connections and brake adjustments are the responsibility of the user and do not constitute grounds for complaint.

Maintenance schedule:

Every week Check the brake for failure, check all wheels for loose bolts, nuts and axles

Periodically Check the brackets for wear, check that the frame bolts are tight, check that the front and rear axle bolts are tight, check the "fatigue" of the components for wear or cracks.

Troubleshooting:

If you have any problems, please review this section. If this does not help, please contact your point of sale or service. Repairing the stand yourself is PROHIBITED!!! This can cause an accident.

I. The stand is not stable.

- a. Are both support handles (1) at the same height? If not, adjust them correctly.
- b. Are the clamps/screws (6) securely tightened/tightened? If not, tighten/tighten them.
- c. Is the kickstand fully extended? If not, extend it and check its stability.
- d. Are all the wheels (5) and (8) tightened securely? If not, tighten them securely.
- e. Are all the parts of the support frame (7) securely connected? If not, contact your point of sale. DO NOT USE THE SUPPORT!!!

Incidents:

Everyone **serious** an incident related to the product should be reported to the manufacturer, the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

Utilization:

A product that is no longer suitable for use must be disposed of in accordance with local environmental regulations regarding product disposal.

Discontinuation of use of the product:

In case of visible mechanical damage or material wear, stop using the product.

COMPONENTS / USEFUL ELEMENTS

ATTENTION!
NEVER STAND ON THE FOOTREST (9)
RISK OF ACCIDENT!



1. Handles / Push handles
2. Brake handles with parking function
3. Shopping basket/bag
4. Seat
5. Wheels with brake
6. Clamps / Screws / Handle Latches
7. Frame
8. Steering wheels (fixed AR-020, AR-021)
9. Footrest
10. Backrest
11. Rubber feet

Preparation for use

1. Remove the support from the cardboard or foil packaging.
2. First of all, check the technical condition of the support in accordance with point Notes and warnings.
3. Insert the handles and push handles (1) into the holes in the frame (7). See the drawing above.
4. Clamp/screw the handles and push handles (1) to the frame (7) in such a way that they cannot be removed.
5. Adjust the height of the support and push handles (1) to the user's height. To do this, unscrew/remove the clamp (AR-005), screw (AR-006, AR-020, AR-021, AR-022), latch (AR-007, AR-032) (6) located in the frame (7). Set the required height by sliding or sliding them together. Reinsert the screw/latch into the existing hole in the frame, tighten/tighten. Repeat the operation for both support handles. The pushing handles (1) must be at the same height.
6. After unfolding, it is absolutely necessary to check the correct operation of the brakes (does not apply to AR-020, AR-021). Place your hands on the handles (1), pull the brake handles up with your fingers (2) and then check whether the clamps block the wheels with the brakes (5). Additionally, after the test, push the brake handles (2) with your fingers as far down as possible until the latch "clicks". The brake should be in the parking position and then check by gently pushing the walker forward whether the wheels are blocked.
7. Install the basket, tray (AR-005, AR-022), bag (AR-007, AR-032) (3). See the drawing above.

Alternatively, please consult your doctor, physical therapist or point of sale for precise instructions on appropriate use.

End of Use:

1. Fold the stand by sliding the front wheels and rear wheels/legs towards each other (AR-005, AR-006, AR-020, AR-021, AR-022).
2. In the AR-006, AR-020, AR-021, AR-022, AR-032 model, the folding handle is located under the seat. Fold the support (AR-007) by sliding the right and left wheels towards each other by pulling the handle in the seat.

Cleaning and storage:

1. Place the support in a place where it will not be exposed to mechanical damage.
2. In case of dirt, refer to the "Maintenance Method" section of this manual.

THANK YOU FOR CHOOSING AND PURCHASING OUR PRODUCT